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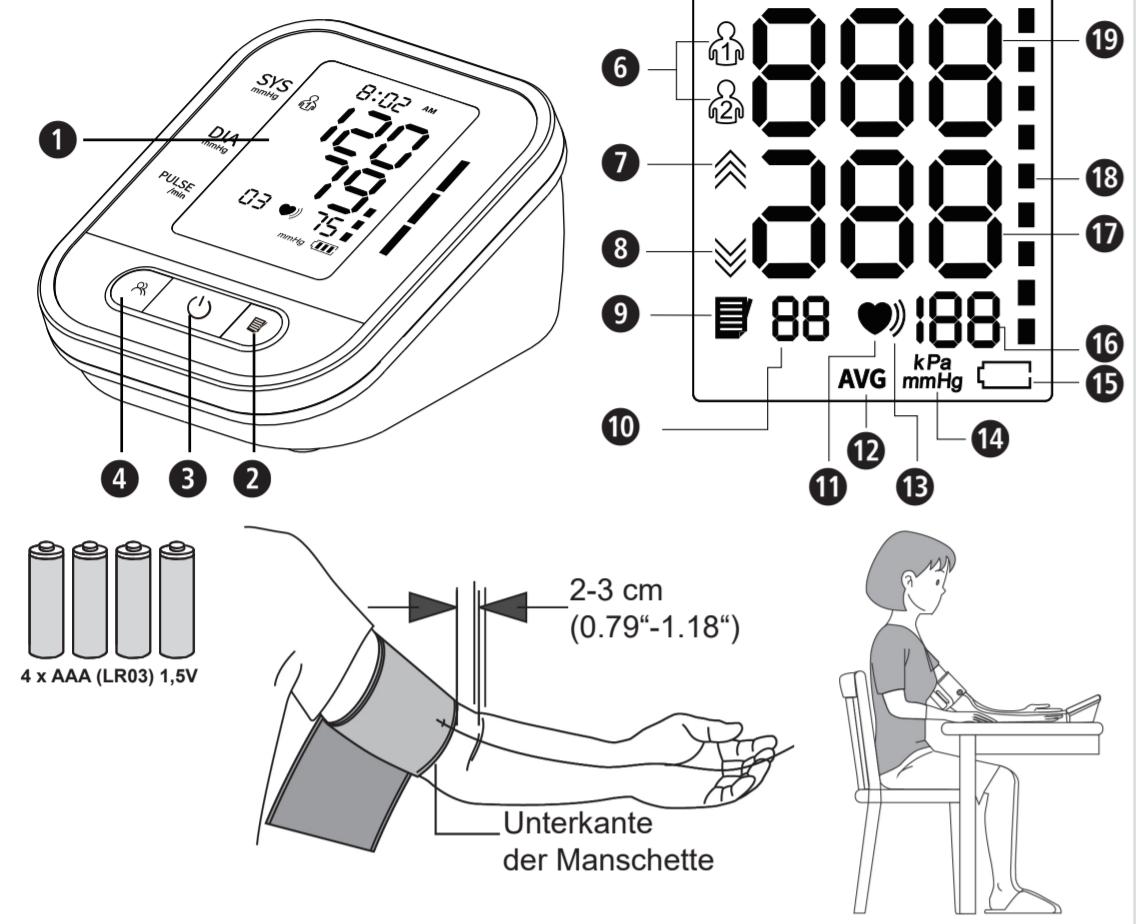
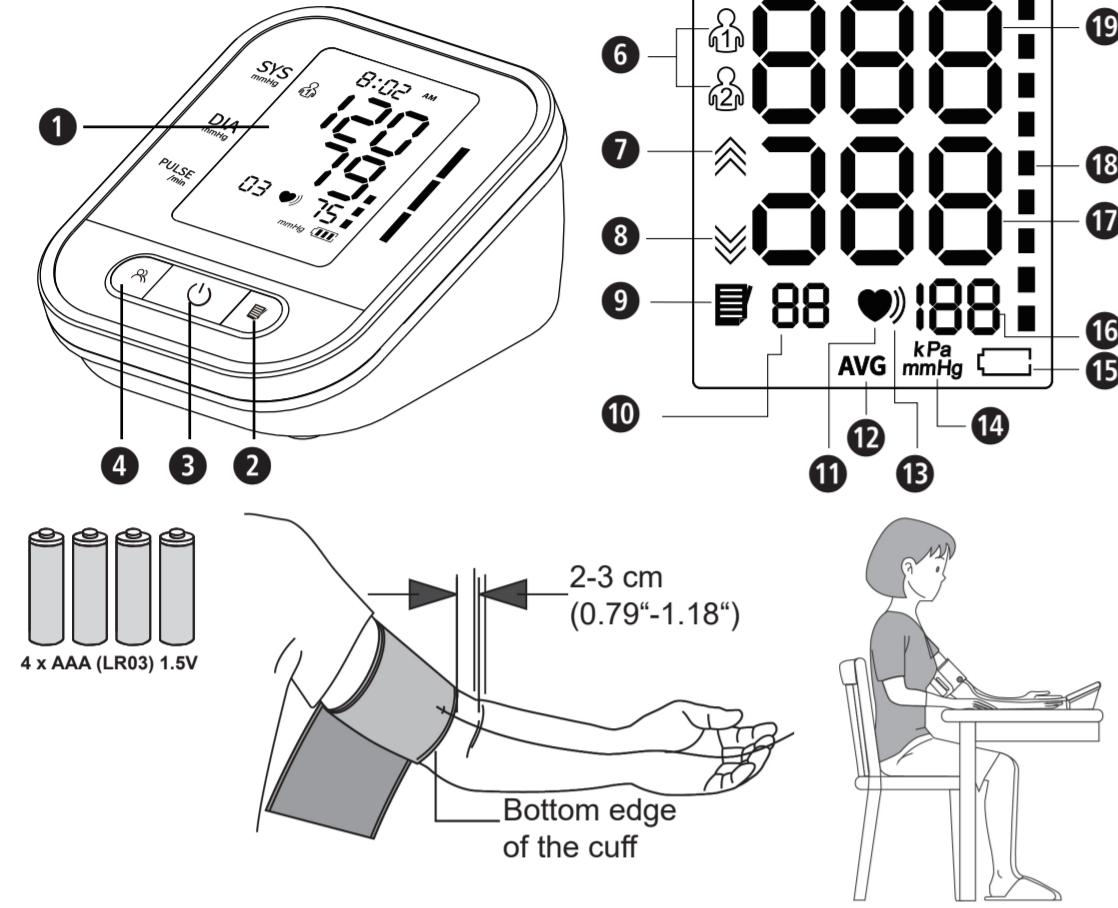
Blood pressure monitor BU-95E

Instruction manual - Please read carefully!

GB Unit and LC-Display

Intended use

- This upper arm blood pressure monitor is designed to measure the systolic and diastolic pressure and pulse rate of an adult person using non-invasive oscillometric technique through an inflatable cuff placed around the upper arm. The device can be used at home.

Contraindications

- There are no known contraindications.

Explanation of symbols

IMPORTANT
Follow the instructions for use!
Non-observance of these instructions can result in serious injury or damage to the device.

WARNING
These warning notes must be observed to prevent any injury to the user.

CAUTION
These notes must be observed to prevent any damage to the device.

NOTE
These notes give you useful additional information on the installation or operation.

Device classification: type BF applied part

Battery change symbol

Keep dry

LOT LOT number

Manufacturer

Date of manufacture

Observe local disposal guidelines!

Importer

Distributor

Shenzhen AOJ Medical Technology Co., Ltd.
Add: Room 301&4F, Block A, Building A, Jingfa Intelligent Manufacturing Park, Xiwaiyuan, Gushu Community, Xixiang Street, Bao'an District, 518126 Shenzhen, CHINA

Share Info GmbH
Address: Heerdrter Lohweg 83, 40549 Düsseldorf, GERMANY

Tel: 0049 179 5666 508; E-mail: EU-Rep@share-info.com

EC | REP EU authorized representative

GB Safety Instructions

Read the instruction manual carefully before using this device, especially the safety instructions and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- Only use the unit for its intended purpose in accordance with the instructions for use. Warranty claims become void if the unit is misused.
- If you have any medical conditions (such as arterial occlusive disease, arrhythmia, atrial or ventricular atrial fibrillation, arteriosclerosis, poor circulation, diabetes, pre-eclampsia or kidney disease), consult your doctor before using the device. These diseases - just like movements, tremors or shivering - can lead to false readings.
- The unit may not be used to check the heart rate of a pacemaker.
- The result of a blood pressure measurement with this device is not suitable for self-diagnosis or self-treatment. To avoid health risks, always follow your doctor's instructions and contact him/her for an assessment of the measurement result.
- Never change the intake of medication prescribed by a doctor as a result of a measurement with this device and consult your doctor for advice on the best time to take the measurement before measuring.
- The device is not suitable for continuous monitoring of a patient, assessment of an existing disease or first aid.
- The device is not suitable for use on patients undergoing surgical treatment and should only be used in specified locations or in private homes.
- Pregnant women should take the necessary precautions and pay attention to their individual circumstances. Please consult your doctor if necessary.
- The patient is an intended operator and all the functions can be safely used.
- If discomfort such as upper arm pain, skin irritation or other discomfort occurs during a measurement, press the start/stop button **②** to deflate the cuff immediately. Loosen the cuff and remove it from the upper arm.
- This device is not designed to be used by persons with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- This device is intended to be used by adults. The unit is unsuitable for children or babies. Contact your doctor, if you want to use this device on young persons. Medical devices are not toys!
- Keep the appliance out of the reach of children.
- The device must not be operated in rooms with high radiation levels or in the vicinity of devices with high radiation levels such as radio transmitters, mobile phones or microwaves or in the vicinity of HF surgical devices, MRI devices (magnetic resonance imaging) or computer tomographs. This may cause malfunctions or incorrect readings.
- Protect the unit from moisture. Should liquid nevertheless penetrate the device, the batteries must be removed immediately and further applications avoided. In this case, contact your specialist dealer or inform us directly.
- Never use the unit outdoors or in damp rooms.
- Never use the unit in moving vehicles (e.g. cars).
- Do not operate the unit near flammable gas (e.g. narcotic gas, oxygen or hydrogen) or flammable liquid (e.g. alcohol).
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.
- Repeated blood pressure measurements in high numbers can lead to undesirable side effects, e.g. nerve crushing or blood clots.
- In case of performance fluctuations (e.g. inaccurate readings or abnormal display), stop using the device and contact customer service.
- Never place the cuff over injured skin, when a catheter is in place, or during blood transfusions.
- Talk to your doctor before using this device on an arm where there is an intravascular access or therapy or an arterio-venous (A-V) shunt, as there may be a temporary disruption of blood flow which could result in injury.
- Talk to your doctor before using the device if you have had a mastectomy or suffer from circulatory problems or blood disorders, as inflating the cuff may cause bruising.
- To avoid strangulation, always keep the cable and air tube away from children, infants and young children.
- Do not kink or bend the air hose excessively.
- Keep the device away from heavy shocks or vibrations and do not drop it. Never use force, e.g. when putting on the cuff.
- Only inflate the cuff when it is properly applied to the upper arm.
- Remove the cuff if the air is not automatically pumped out during measurement.
- Use only original accessories and spare parts from the manufacturer, otherwise damage to the unit or personal injury may result.
- Observe the local disposal guidelines if you wish to dispose of the unit.
- If you have any doubts about the proper functioning of the unit, stop using it and contact the service centre immediately.
- Swallowing small parts such as packaging material, battery, battery compartment cover, etc. can lead to suffocation. Therefore, never leave children alone with the unit.
- No maintenance or cleaning may be carried out during operation. Follow the maintenance and cleaning instructions in this manual.
- Do not measure blood pressure when other measurements are being taken on the same part of the body at the same time, as these may be disturbed or fail.
- If the ambient temperature is less than 5°C or more than 40°C, the waiting time from minimum storage temperature to use is at least 1 hour; the time from maximum storage temperature is at least 2 hours.
- Remove the batteries if you are not using the device for a long time.

SAFETY NOTES FOR BATTERIES

- Do not disassemble batteries!
- Replace all batteries if the unit display shows the low battery symbol.
- Never leave any low battery in the battery compartment since it may leak and cause damage to the unit.
- Increased risk of leakage! Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed, seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Remove the batteries from the device if it is not going to be used for an extended period of 3 months or more!
- Keep batteries out of children's reach!
- Do not attempt to recharge batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

Unit and LC-Display

- | | | | |
|--|--|-------------------------------------|-------------------------|
| ① LC-Display | ② User memory | | |
| ③ Start/Stop | ④ User selection / settings | | |
| ⑤ Time/Date | ⑥ User icon 1/2 | ⑦ Inflation icon | ⑧ Deflation icon |
| ⑨ Memory icon | ⑩ Memory number | | |
| ⑪ Pulse icon | ⑫ Average value | ⑬ Symbol irregular heartbeat | |
| ⑭ Blood pressure measuring unit | ⑮ Battery change icon | | |
| ⑯ Display of Pulse Rate | ⑰ Display of Diastolic Pressure | | |
| ⑲ Blood pressure indicator | ⑳ Display of Systolic Pressure | | |

Scope of delivery

First check that the unit is complete. The scope of delivery includes:

- 1 ecomed Blood pressure monitor BU-95E
- 4 Batteries (type AAA, LR03) 1.5V
- 1 Cuff 22-42 cm
- 1 Instruction manual

If you notice any transport damage when unpacking, please contact your dealer immediately.

WARNING

Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!

What is blood pressure?

Blood pressure is the pressure created in the blood vessels by each heart beat. When the heart contracts (= systole) and pumps blood into the arteries, it brings about a rise in pressure. The highest value is known as the systolic pressure and is the first value to be measured. When the cardiac muscle relaxes to take in fresh blood, the pressure in the arteries also falls. Once the arteries are relaxed, a second reading is taken – the diastolic pressure.

Blood pressure classification

Low blood pressure	systolic <100 diastolic <60
Normal blood pressure	systolic 100 - 139 diastolic 60 - 89

Forms of hypertension

Mild hypertension	systolic 140 - 159 diastolic 90 - 99
Moderate hypertension	systolic 160 - 179 diastolic 100 - 109
Severe hypertension	systolic ≥ 180 diastolic ≥ 110

WARNING

Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Do not make any modifications to the unit, do not disassemble it and never repair the unit yourself. Have repairs carried out only by authorised service centres.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least 5-10 minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.
- Repeated blood pressure measurements in high numbers can lead to undesirable side effects, e.g. nerve crushing or blood clots.
- In case of performance fluctuations (e.g. inaccurate readings or abnormal display), stop using the device and contact customer service.
- Never place the cuff over injured skin, when a catheter is in place, or during blood transfusions.
- Talk to your doctor before using this device on an arm where there is an intravascular access or therapy or an arterio-venous (A-V) shunt, as there may be a temporary disruption of blood flow which could result in injury.
- Talk to your doctor before using the device if you have had a mastectomy or suffer from circulatory problems or blood disorders, as inflating the cuff may cause bruising.
- To avoid strangulation, always keep the cable and air tube away from children, infants and young children.
- Do not kink or bend the air hose excessively.
- Keep the device away from heavy shocks or vibrations and do not drop it. Never use force, e.g. when putting on the cuff.
- Only inflate the cuff when it is properly applied to the upper arm.
- Remove the cuff if the air is not automatically pumped out during measurement.
- Use only original accessories and spare parts from the manufacturer, otherwise damage to the unit or personal injury may result.
- Observe the local disposal guidelines if you wish to dispose of the unit.
- If you have any doubts about the proper functioning of the unit, stop using it and contact the service centre immediately.
- Swallowing small parts such as packaging material, battery, battery compartment cover, etc. can lead to suffocation. Therefore, never leave children alone with the unit.
- No maintenance or cleaning may be carried out during operation. Follow the maintenance and cleaning instructions in this manual.
- Do not measure blood pressure when other measurements are being taken on the same part of the body at the same time, as these may be disturbed or fail.
- If the ambient temperature is less than 5°C or more than 40°C, the waiting time from minimum storage temperature to use is at least 1 hour; the time from maximum storage temperature is at least 2 hours.
- Remove the batteries if you are not using the device for a long time.

Starting up: Insert / replace batteries

Before you can use your device, you must insert the enclosed batteries. The cover of the battery compartment is located on the back of the unit. Open it and insert 4 1.5V batteries, type AAA LR03. Pay attention to the polarity (as marked in the battery compartment). Close the battery compartment again.

Replace the batteries immediately when the battery change icon **⑭** appears in the display or when nothing appears in the display after the unit has been switched on.

User setting Briefly press the **④** - button **②** to change the user setting. Pressing the **④** - button **②** briefly again switches between users 1 or 2.

Setting the date and time

The upper arm blood pressure monitor automatically stores up to 90 readings for each user (1 or 2). The memory function stores the readings with the date and time.

Set the device to the current date and time before taking a measurement for the first time. The date and time cannot be changed during a measurement.

1. Press and hold button **④** until the year flashes.

2. Use the button **②** to set the year and then press **④** again to move to the month setting.

3. Now use the button **②** again to set the month and do the same for the day.

4. After date setting, the time needs to be set. Shortly press button **②** to switch the time system (12 or 24 hour display) and choose the time system you want.

5. After setting the time system, the computer will automatically enter the time setting mode.

Adjust the hours and minutes by using the buttons **②** and **④**.

2. Finally, briefly press button **④** to confirm the setting.

Setting the measuring unit

The unit supports measurement in mmHg (default setting) or in kPa (selectable).

To change the measuring unit, press and hold the Start/Stop button **③** for approx. 10 seconds and then set the desired unit with the button **②**. Finally, briefly press button **④** to confirm the setting.

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